



AED Monthly Checklist

15 Feb 2021

Complete

| Score | 81.25% | Failed items | 1 | Actions | 2 |
|-----------------|--------|--------------|---|---|-------------|
| Location of AED | | | | 2114 Central St 6th floor, Kansas City, MO 64108, USA (39.0867543, -94.5879528) | |
| AED Serial No. | | | | | 12345678910 |
| Checked on | | | | 15th Feb, 2021 7:15 AM PST | |
| Checked by | | | | SafetyCulture Staff | |

Failed Items & Actions

1 failed, 2 actions

Failed items

1 failed, 0 actions

AED Monthly Checklist

Are the electrode pads in the AED container?

No



Photo 2

Other actions

2 actions

AED Monthly Checklist / Check PPE/Ready Kit

Scissors

To do | Priority Medium | Due 19th Feb, 2021 8:00 AM PST | Created by SafetyCulture Staff

Buy scissors and place in AED ready kit.

AED Monthly Checklist

Test AED container tamper alarm

In progress | Priority High | Due 15th Feb, 2021 8:00 AM PST | Created by SafetyCulture Staff

Test AED container tamper alarm.

AED Monthly Checklist

2 actions, 1 failed, 80%

Is the AED secure in its container?

Yes



Photo 1

- Check AED is on standby mode - green flashing light

If AED is not on standby mode, verify AED operation

Verify AED Operation

100%

- Open AED lid / Turn on AED
- Wait for status indicator to change from RED to GREEN after five seconds
- Close AED lid / Turn off AED
- Wait for status indicator to change from RED to GREEN after five seconds

Are the electrode pads in the AED container?

No



Photo 2

- Check expiration date of electrode pads

Electrode Pads Expiration Date:

22nd Feb, 2021

Is there an extra set of electrode pads?

Yes

- Check expiration date of AED battery

The AED battery is expired if 4 years have passed since its installation.

AED Battery Installation Date:

14th Feb, 2020

Check PPE/Ready Kit

1 action, 75%

- Mask
- Scissors

To do | Priority Medium | Due 19th Feb, 2021 8:00 AM PST | Created by SafetyCulture Staff

Buy scissors and place in AED ready kit.

Razor

Gloves

Test AED container tamper alarm

In progress | Priority High | Due 15th Feb, 2021 8:00 AM PST | Created by SafetyCulture Staff

Test AED container tamper alarm.

IMPORTANT Maintenance

100%

If electrode pads were expired, did you replace them?

N/A

Electrode pads not expired.

Input the date the electrode pads were replaced:

16th Dec, 2020

If AED battery was expired, did you replace it?

N/A

AED battery not expired.

Input the date the AED battery was replaced:

14th Feb, 2020

If the AED container tamper alarm was not working, did you change its battery?

Yes



Photo 3

Signature

A handwritten signature in black ink, appearing to be "AED Master".

AED Master

15th Feb, 2021 7:41 AM PST

Appendix

Accessibility



Photo 1



Photo 2



Photo 3