



AED Maintenance Checklist

10 Apr 2024 / Hester funk

Complete

Score	12 / 13 (92.31%)	Flagged items	1	Actions	0
--------------	------------------	----------------------	---	----------------	---

Checked on

10 Apr 2024 12:30 AEST

Checked by

Hester funk

Flagged items

1 flagged

AED Maintenance Checklist / AED Maintenance Log

Batteries within expiration date. Replace if expired.

Fail

Batteries replaced

AED Maintenance Checklist

1 flagged, 12 / 13 (92.31%)

AED Maintenance Checklist

6 / 6 (100%)

Inspect frequently, as necessary.



Check for the green check showing that the AED is ready to use.



Verify that electrodes are within their expiration date.



Electrode expires on January 2025



Photo 1

Verify that batteries are within their expiration date.



Verify that electrodes are pre-connected to the input connector.



Verify that supplies are available for use (razor, mask, gloves, extra batteries.)

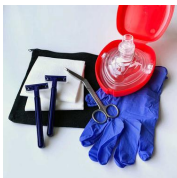


Photo 2

AED Maintenance Log

1 flagged, 6 / 7 (85.71%)

Is the unit clean, undamaged, free of excessive wear?

Pass

Are there any cracks or loose parts in the housing?

Pass

Verify electrodes are connected to the AED and sealed in their package. Replace if expired.

Pass

Electrode pads expires on January 2025

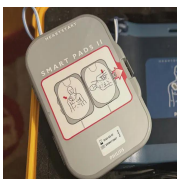


Photo 3

Are all cables free of cracks, cuts and exposed or broken

Pass

wires?



Turn the AED on and off and verify the green check indicates ready for use.



Batteries within expiration date. Replace if expired.



Batteries replaced

Check for adequate supplies.



Media summary



Photo 1



Photo 2



Photo 3